

Holter Intialize

1. Put holter disc in SanDisk next to the front desk holter computer.
2. Put in lower slot with arrow upper side pointing in.
3. Green light will come on SanDisk if put in properly
4. Exit out
5. Go to desk top screen click into Vision series holter system.
6. Press on initialize pc card.
7. Pick 2 channel and 24/48 hour.
8. Type pt info and date etc.
9. press ok, check press ok again.

Acquire Holter

1. Put the returned holter disc into the SanDisk lower slot as mentioned above.
2. Click on print.
3. Click on specific page
4. Click or mark cover page ok.
5. Then print.