

# STEP-BY-STEP for Holter Performer PLUS Set-Up Procedure

Remove the **Black Plug** from the Holter Performer PLUS (HPP) Recorder.  
Wait 3 seconds until it beeps



Insert the HPP Recorder into the Docking Station.




Launch Master 9 or ACSD 9 Software and  
**Select Option 3**  
<Setup Holter Performer Plus>



Select <**Event Mode**> or <**Holter mode**>,  
then Enter the Patient ID,  
then **Disable** Audible Alert,  
then select the <**Write to Device**> tab.  
Select <**Exit**>.

Remove the recorder from the Docking Station.

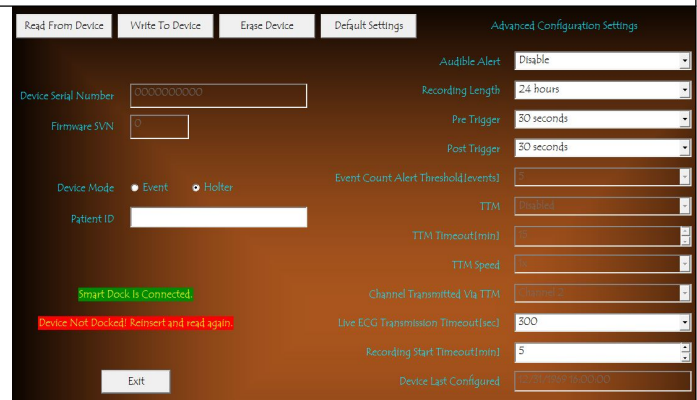

Hook up the patient with the lead set disconnected.  
After the leads are in place, press and HOLD the **Record** button for 3 seconds.  
Once the Green light goes on and 1 BEEP is heard, the monitor is recording and working properly.

